

RENASYS High Output/Fistula Kit

Effective management of complex, high-output explored enterocutaneous fistula wounds

Negative Pressure Wound Therapy (NPWT) has become an established method for advanced wound care. Smith & Nephew provides a NPWT solution specifically designed for one of the most complex wounds: high-output explored enterocutaneous fistulas.

The RENASYS High Output/Fistula Kit from Smith & Nephew, in conjunction with the RENASYS EZ Plus pump, is an intuitive, cost-effective, and clinically proven NPWT solution for managing explored enterocutaneous fistulas while enhancing patient comfort throughout the healing process.

The RENASYS High Output/Fistula Kit can be used with the RENASYS EZ Plus device, a powerful NPWT pump that is compatible with all RENASYS dressing kits. This gives you the flexibility to leverage your RENASYS equipment across multiple care settings, patients and wound types.



A 56-year-old female with a dehiscenced surgical wound and small bowel fistula was treated with gauze-based NPWT at -80 mm Hg with dressing changes three times per week due to high levels of effluent. Result: 92.2% resolution in 72 days.



The RENASYS[◇] High Output/Fistula Kit enhances ease of use and helps clinicians effectively manage complex high-output fistula wounds

Enhanced flexibility

- Tight dressing seals ensure continuous, efficient exudate removal
- All-in-one kit contains application specific components to help manage complex high-output fistulas
- RENASYS EZ Plus pump is compatible with all RENASYS dressing kits allowing the equipment to be leveraged across multiple care settings, patients and wound types.

Exceptional clinical and service support

- 24/7 hotline for clinical consultation on NPWT and patient cases
- Clinical education and training for appropriate NPWT utilization

RENASYS Ordering Information

Dressing Kits	
Product Description	Product Code
RENASYS High Output/Fistula Kit	Smith & Nephew 66800932

For more information and ordering,

Call **US** 1-800-876-1261

Call **Canada** 1-800-463-7439

With over 150 years of experience in advanced wound care, Smith & Nephew is an industry leader providing innovative solutions to meet the needs of chronic, acute and traumatic wounds across all care settings.

Wound Management Smith & Nephew, Inc.

USA
970 Lake Carillon Drive
Suite 110
St. Petersburg, FL 33716

Customer Care Center
1-800-876-1261
T 727-392-1261
F 727-392-6914

Canada
2250 Alfred-Nobel Blvd.
Suite 300
St-Laurent, Quebec
H4S 2C9

Customer Action Center
T 1-800-463-7439
F 1-800-671-9140

www.smith-nephew.com
www.myrenasys.com

References

1. Chariker ME, Jeter KF, et al. Effective management of incisional and cutaneous fistulae with closed suction wound drainage, Contemporary Surgery. 1989; 34(1): 59-63.
2. Terri Reed, BSN, WOCN; Diana Economon, RN, BSN, CWOON; and Laurel Wiersema-Bryant, APRN, BC Colocutaneous Fistula Management in a Dehisced Wound: A Case Study; Ostomy Wound Management 2008.
3. McCord SS, et al. Negative pressure therapy is effective to manage a variety of wounds in infants and children, Wound Repair and Regeneration. 2007.
4. Panicker VN. A pilot study evaluating topical negative pressure using VISTA[®] technology. Wound Practice and Research. Vol 17, No. 4, November 2009.
5. Leak K., Johnson, S., Negative Pressure Wound Therapy: a cost effective option, Presented at Symposium on Advances in Skin and Wound Care (SAWC) 2008.
6. Morehouse T, Hager J, Drainage management of enterocutaneous fistula within an abdominal wound, Presented at Symposium on Advances in Skin and Wound Care (SAWC) 2008.
7. Howe T, Graham R, The successful management and wound closure of a challenging patient with Necrotizing Fasciitis with Enteric Fistula using NPWT, Presented at Symposium on Advances in Skin and Wound Care (SAWC) 2008.
8. Campbell P, Smith G, Smith J, Martin R Retrospective evaluation of a gauze-based wound filler. Presented at the World Union of Wound Healing Societies (WUWHS) 2008.
9. Murphy W, Madura J, Negative Pressure Wound Therapy for Fistula Management. Presented at Wound, Ostomy and Continence Nurses Society (WOCN) 2008.
10. Blevins A, Management of four patients with complicated enterocutaneous fistula using pouching system in conjunction with negative pressure therapy in a long term acute care facility (LTAC), Presented at Wound, Ostomy and Continence Nurses Society (WOCN) 2008.
11. Dube MK, Malick HI, McAdam JG, Management of a complex necrotizing fasciitis wound in district general hospital, WOUNDS UK 2009.
12. Wharton S, Management of complex acute wounds in a district general hospital with Negative Pressure Wound Therapy (NPWT), WOUNDS UK 2009.
13. Reed T, Economon D, Successful patient care of large soft tissue defects-the Barnes-Jewish Hospital experience. Presented at Symposium on Advanced Wound Care (SAWC) 2010.
14. Blevins A, Using different dressing techniques with Versatile Negative Pressure Wound Therapy (VNPWT). Presented at Symposium on Advanced Wound Care (SAWC) 2010.
15. Zanotti E, Newman K, High output fistulas within abdominal wounds – the ultimate challenge, Presented at Wound, Ostomy and Continence Nurses Society (WOCN) 2010.